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
Dear Customer,

With the Acculan system you have also obtained a very powerful battery with integrated electronic. Due to its construction and the computerized charging unit the battery has a very long life expectancy if it is treated properly.

Therefore we would ask you to observe the following instructions:

CAUTION!

- The temperature of the battery should not exceed 65 °C.
If the battery is heated over this temperature, the battery loses capacity, and it will even be destroyed if the heating lasts longer.
- Avoid a short-circuit.
If the poles of the battery (either top side or down side) are connected by metal or liquid, high current will flow (comparable with a car battery). The battery may be heated up extremely and be destroyed and it may even damage surrounding objects (danger of fire).

Therefore we recommend to treat the battery as follows:

- Store the battery at room temperature (about 20 degrees celsius).
- Do not sterilize or disinfect the battery by any means.
As the battery is completely sealed inside the housing of the machine the contamination of the OR-field by the battery is not possible.
- Always keep the battery dry.
- Never immerse the battery in fluids.
- Never place the battery on wet surfaces.
- Always make sure that the sockets of the charging unit are dry inside before you put the battery in.
- Do not place or run the charging unit near to places where you handle fluids (instrument cleaning, infusions).

If you follow these instructions, we assure you will have many successful operations with your Acculan system.

If you have any additional questions, please call your local AESCULAP sales representative.

Keep dry

Recycable

Disposal

The NiCd cells of rechargeable batteries GA 626 and GA 646 contain heavy metals and may not be disposed of with conventional refuse.

- The NiCd cells of rechargeable batteries GA 626 and GA 646 should be recycled in an environmentally safe manner.

The NiMH cells of rechargeable battery GA 616 contain no heavy metals.

- The NiMH cells of rechargeable battery GA 616 may be disposed of with conventional refuse.

Application

For information regarding the use of the rechargeable batteries, please consult the appropriate instructions for use.

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AESCULAP AG & CO. KG