

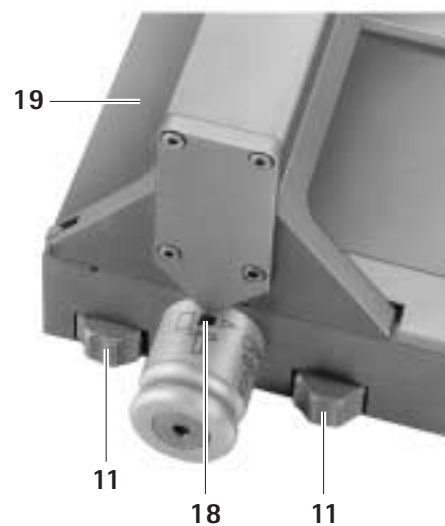
Aesculap Power Systems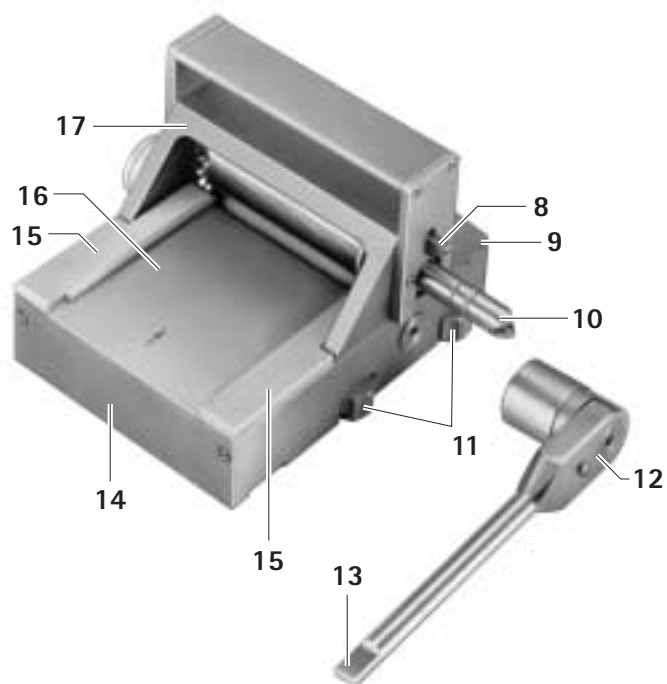

-
- (GB)** Instructions for use/Technical description
Skin mesh dermatome BA720R
 - (D)** Gebrauchsanweisung/Technische Beschreibung
Hautnetz-Dermatom BA720R
 - (F)** Mode d'emploi/Description technique
Dermatome pour trame dermique BA720R
 - (E)** Instrucciones de manejo/Descripción técnica
Dermatomo para injertos de malla BA720R
 - (I)** Istruzioni per l'uso/Descrizione tecnica
Estensore di reticoli cutanei BA720R

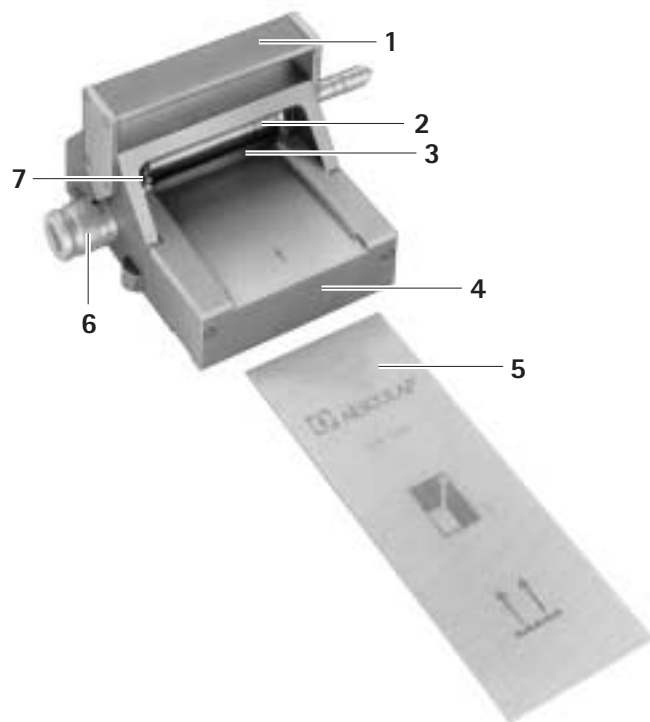
Skin mesh dermatome BA720R

Legend

- 1 Carrying handle
- 2 Cutting cylinder
- 3 Counter cylinder
- 4 Cutting device
- 5 Carrier plate for skin graft
- 6 Setting wheel for cutting depth
- 7 Gearwheel transmission
- 8 Cutting cylinder lock
- 9 Hinge
- 10 Shaft of cutting cylinder 2
- 11 Fastening screws for upper part 17
- 12 Hand ratchet for driving the cutting cylinder
- 13 Screwdriver end for screws 11 and shaft 10
- 14 Lower part
- 15 Guide strips for carrier plates
- 16 Insertion tray for carrier plates 5
- 17 Upper part, hinged
- 18 Scale of the setting wheel
- 19 Removal tray for carrier plates
- 20 Bearings

Symbols on the product

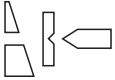	Attention, see instructions for use
	Scale for "Skin graft cutting"
	Scale for "Skin graft scoring"

Contents

1. Safe handling	2
2. Product description	3
2.1 System components	3
2.2 Intended use	3
2.3 Mode of operation	3
3. Preparation and setup	3
4. Working with the skin mesh dermatome BA720R	3
4.1 System set-up	3
4.2 Function check	3
4.3 Safe operation	3
5. Care and handling	6
5.1 Manual disinfection and cleaning	6
5.2 Mechanical cleaning and disinfecting	6
5.3 Care and inspection	6
5.4 Sterilization method and parameters	6
6. Maintenance	6
7. Troubleshooting list	7
8. Technical Service	7
9. Accessories/Spare parts	7
10. Technical data	7
11. Disposal	7

1. Safe handling

- Prior to use, check for proper condition and functioning of the product.
- To avoid damaging the product through improper setup or operation and mitigating the warranty and liability on the part of the product's manufacturer:
 - Use the product only according to these instructions for use.
 - Follow the safety instructions and maintenance advisories.
 - Only combine Aesculap products with each other.
- Only have the products and accessories operated and used by persons who possess the requisite training, knowledge or experience.
- Keep the instructions accessible for the O.R. staff.

2. Product description

2.1 System components

Designation	Art. no.
Skin mesh dermatome, complete with hand ratchet	BA720R
Carrier plate for expansion factor 1.5; sterile	BA721
Carrier plate for expansion factor 3; sterile	BA722
Carrier plate for expansion factor 6; sterile	BA723

2.2 Intended use

The skin mesh dermatome is used for the cutting and scoring of skin grafts in general, burns and plastic surgery.

2.3 Mode of operation

The cutting cylinder **2** is turned anticlockwise with hand ratchet **12**. Carrier plate **5** transports the skin graft between cutting cylinder **2** and counter cylinder **3**. During this transport, the skin graft is scored or cut. The cutting depth is set with setting wheel **6**, using scale **18**. The staggered slits in the skin graft allow expanding the graft to rhombic skin mesh. The skin mesh can be used for covering defects of up to six times the size of the grafting site.

3. Preparation and setup

Non-compliance with these rules will result in complete exclusion of liability on the part of Aesculap.

- Observe the following rules when setting up and operating the product:
 - Prior to use, inspect the skin mesh dermatome BA720R and its accessories for visible damage.
 - Only use the skin mesh dermatome BA720R and its accessories if they are in perfect condition.
 - Be certain that the skin mesh dermatome is set up on a sufficiently stable support (e.g. table, equipment cart, etc.).

4. Working with the skin mesh dermatome BA720R

4.1 System set-up

Risk of infection and contamination!

The device and its accessories are delivered in unsterile condition.

- Sterilize the device and accessories before use.

- Slide hand ratchet **12** on shaft **10** so that it clicks into position.

4.2 Function check

Carry out a trial run of the skin mesh dermatome BA720R prior to each use:

- Verify that gearwheel transmission **7** is moved when turning hand ratchet **12**.
- Make certain that setting wheel **6**, once engaged, can be easily rotated through the range of scale **18**.
- Only use the device if it is in perfect condition.

4.3 Safe operation

Setting the cutting cylinder

Note

We recommend using a separate cutting cylinder for scoring the skin graft. This cutting cylinder should not be used for cutting, since that would negatively affect the quality of the scoring results.

Damage to the cutting cylinder! The cutting cylinder can only be set when running freely.

- **Make certain that the carrier plate is not engaged under the cutting cylinder.**
- Slightly pull out setting wheel **6** until it disengages.

Skin mesh dermatome BA720R

- Turn setting wheel **6** to the required position.

Possible settings at the setting wheel	Scale symbols
Skin graft cutting	
extremely thick skin graft	
normal setting for cutting	
reserve setting for cutting	
Skin graft scoring	
thin skin graft	
thick skin graft	

- Slightly push in setting wheel **6** until it engages.

Processing the skin graft

Notes

Only place individual carrier plates on the insertion tray, one after the other.

For long skin grafts, sequentially insert several carrier plates.

Coiling of the skin graft! The skin graft can be lifted off from carrier plate as soon as it has been cut into by the cutting cylinder!

- Press the skin graft onto the carrier plate, using fine tweezers, directly behind the cut-in line.
- If the cutting cylinder has been turned too far already, turn back the cutting cylinder, using the screwdriver end of the ratchet.
- Position the required carrier plate **5**, with the grooved surface facing up, on insertion tray **16**.
- Position the skin graft on carrier plate **5**.
- Push carrier plate **5** between guide strips **15** under cutting cylinder **2** until the cutting cylinder engages.
- Turn hand ratchet **12** anticlockwise.
The carrier plate is transported.
- Keep turning hand ratchet **12** until the complete carrier plate **5** is in position on removal tray **19**.
This completes the processing of the skin graft.

Changing the cutting cylinder

Note

The cutting cylinder cannot be sharpened.

Dismounting the cutting cylinder

Cuts on hands or other body parts, caused by sharp edges on the cutting cylinder!

- Touch the cutting cylinder only at the shaft.

- Turn cutting device **4** upside down so that the four screws **11** are visible, see Fig. 1.
- Loosen screws **11** by hand or, if they are difficult to turn, with the screwdriver end **13** of hand ratchet **12**.
- Return cutting device **4** to its upright position.
- Open upper part **17**, see Fig. 2.
- Hold down lock **8** and, at the same time, pull at shaft **10** to decouple cutting cylinder **2**, see Fig. 3.
- Turn cutting cylinder **2** while pulling it out of bearings **20**, see Fig. 4.
This completes the dismounting of cutting cylinder **2**.

Mounting the cutting cylinder

Note

To prevent transport damage, cutting cylinder **2** is protected by a piece of cardboard. Remove the cardboard protection only after cutting cylinder **2** has been mounted completely.

- Insert cutting cylinder **2** in the two bearings **20**, see Fig. 4.
- Hold cutting cylinder **2** at shaft **10** and press it into bearings **20** until it clicks into position, see Fig. 3.
- Turn shaft **10** and check cutting cylinder **2** for correct seating.
- Close upper part **17** on lower part **14**, see Fig. 2.
- Turn cutting device **4** upside down so that the four screws **11** are visible, see Fig. 1.
- Tighten screws **11** by hand.
- Return cutting device **4** to its upright position.
- Remove the piece of cardboard from cutting cylinder **2**.
This completes the mounting of cutting cylinder **2**.

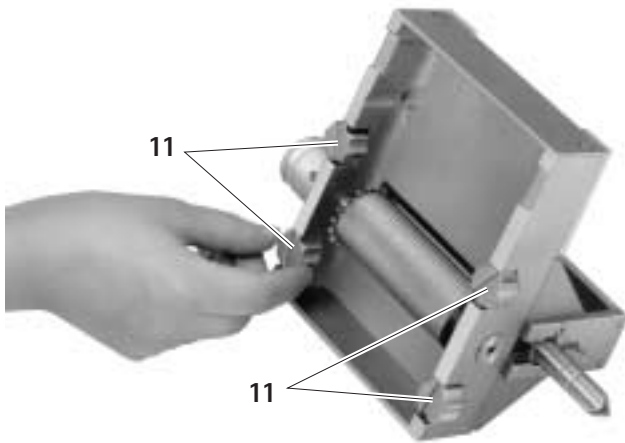
Fig. 1

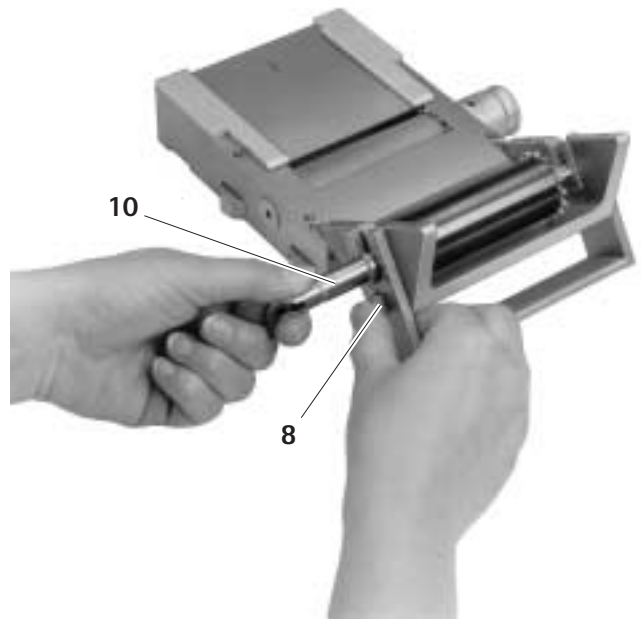
Fig. 3

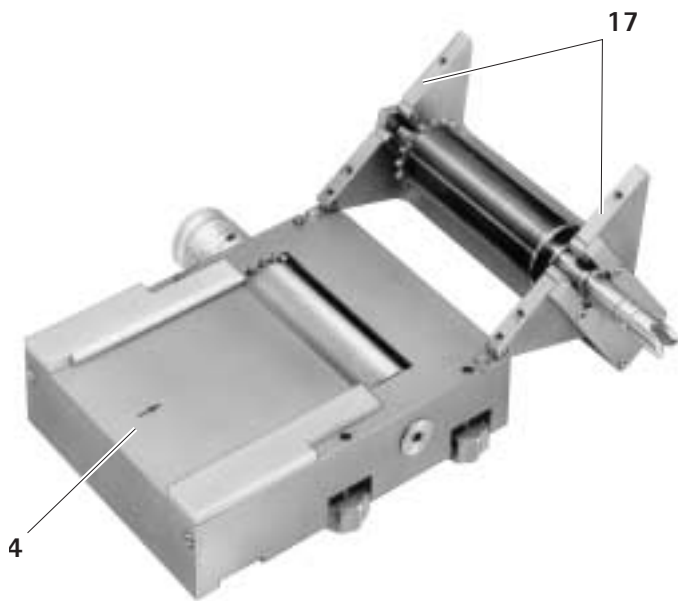
Fig. 2

Fig. 4

Skin mesh dermatome BA720R

5. Care and handling

Note

For patients with Creutzfeldt-Jakob Disease (CJD), suspected CJD or possible variants of CJD, the relevant national regulations concerning the reprocessing of the products must be observed.

5.1 Manual disinfection and cleaning

- Wipe off the product with a clean, lint-free cloth soaked in cleaning and disinfecting agent.
 - or -
 - In order to prevent formation of sticky or encrusted residues, spray the product with a disinfectant spray.
- Wipe off the component surfaces with a clean, lint-free cloth.
- Follow the disinfectant manufacturer's instructions.
- To avoid damage to cutting cylinder 2, clean the cutting cylinder with a soft brush.

5.2 Mechanical cleaning and disinfecting

Damage to the skin mesh dermatome caused by incorrect mechanical cleaning!

- **Only clean the skin mesh dermatome in a suitable storage aid.**
- **Always use a suitable cleaning and disinfection program for mechanical cleaning.**
- **Do not use alkaline cleaning agents.**
- Carry out mechanical cleaning immediately after the operation.
- Install storage aid GB688R in a suitable wire basket (e.g. JF214R).
- Insert the skin mesh dermatome in its proper position in storage aid GB688R immediately after the operation.
- To avoid abrupt changes in temperature, use a single-chamber washer disinfectant.
- Select a cleaning and disinfecting cycle according to the following criteria:
 - Neutral detergent
 - Cleaning temperature ≤ 50 °C
 - Final rinse with simultaneous thermal disinfection with demineralized water

Example: Miele Vario-TD process.

- Follow the instructions for use of the Aesculap ECCOS storage system TA009721.

5.3 Care and inspection

- Allow the product to cool down to room temperature.
- Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functional and undamaged.

5.4 Sterilization method and parameters

Note

The carrier plates BA721, BA722 and BA723 are gamma-sterilized single-use products. These products must not be steam-sterilized and are intended for single use only.

The skin mesh dermatome can be sterilized in its storage aid.

- Sterilize the skin mesh dermatome immediately after cleaning.
- Store the skin mesh dermatome in a suitable wire basket (e.g. JF214R).
- Sterilize with steam, taking note of the following:
 - Carry out steam sterilization via an approved steam sterilization procedure (e.g. sterilizer in conformance with EN 285/ANSI/AAMI/ISO 11134-1993, ANSI/AAMI ST46-1993 and validated in conformance with EN 554/ISO 13683). In the fractioned vacuum process, the sterilization must be performed using the 134 °C/2 bar program with a minimum holding time of 5 minutes.

6. Maintenance

To ensure reliable operation, Aesculap recommends that maintenance be performed after 300 reprocessing cycles or at least once per year.

- For service, maintenance and repairs, please contact the B. Braun/ Aesculap representative in your country, see Technical Service.

7. Troubleshooting list

Error	Finding	Cause	Remedy
Cutting cylinder 2 fails to rotate	Foreign body in device	Cutting cylinder jammed	Remove foreign body
	–	Hand ratchet 12 not engaged	Engage hand ratchet 12
	Excessive axial slack of cutting cylinder	Cutting cylinder loose	Engage cutting cylinder and secure with lock 8
Cutting cylinder 2 moves stiffly	Strong force required on hand ratchet 12	Defective bearing	Repair by manufacturer
Skin not cut through completely	Symbol on scale 18 shows scoring	Incorrect setting	Turn the setting wheel so that symbol "cutting" faces up; select the required fine adjustment, see Safe operation
	Symbol on scale 18 shows cutting; incorrect fine adjustment		Select the required fine adjustment, see Safe operation
	–	Screws 11 loose	Tighten screws by hand or with screwdriver 13
	Cutting tips shiny, hand ratchet 12 stiff	Cutting cylinder 2 worn	Replace cutting cylinder 2 , see Safe operation
	Cutting tips broken off	Cutting cylinder 2 defective	Replace cutting cylinder 2 , see Safe operation

8. Technical Service

For service, maintenance or repairs contact your B. Braun/Aesculap representative.

Carrying out modifications on medical equipment renders the guarantee, and any licenses, null and void.

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 7461 95-2700
Fax: +49 7461 16-2887
E-mail: ats@aesculap.de

Other service addresses can be obtained from the address indicated above.

Designation	Art. no.
Hand ratchet	BA726R
Basket storage aid	BA727R
Storage aid	GB688R

10. Technical data

Model	BA720R
Dimensions (L x W x H)	150 mm x 195 mm x 105 mm
Weight	4.600 g
Length of carrier plate	220 mm
Width of carrier plate	75 mm

9. Accessories/Spare parts

Designation	Art. no.
Replacement cutting cylinder	BA725R
Carrier plate factor 1.5	BA721
Carrier plate factor 3	BA722
Carrier plate factor 6	BA723

11. Disposal

The skin mesh dermatome BA720R is made of aluminum and stainless steel.

- Dispose of the device with other recyclable metals.

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Aesculap AG & Co. KG